


**TECHNICAL SUPPORT:  
877.710.6999**

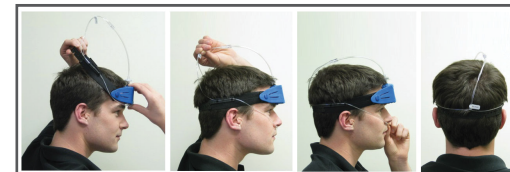

**Step 1** Thoroughly wash and dry your forehead.

**Step 2** Remove the plastic covers from the electrodes.

**Step 3** Hold the cannula tips (2 prongs extending from clear plastic tubing) against the bottom of the Unicorder with your thumb and slide the strap over your head.

**Hold the cannula tips with your thumb**



**Step 5** Place the cannula tips inside your nostrils.

Grasp the cannula with your hand and pull it away from the back of your head until it is snug.

**Place the cannula tips inside your nostrils**



**Step 4** Center the Unicorder over your nose and slightly above your eyebrows. Remove any hair from under the sensors and ensure the sensors and electrodes make complete contact with your skin.

**Slide the strap over your head**



**Step 6** Using your other hand, grab the slip tube and pull toward the back of your head until the tube rests snugly above the plastic portion of the strap. When properly tightened, the cannula cannot be pulled away from your nose.

**Pull the cannula back so that the tips are snug inside your nostrils**

